

THE RESULTS

VESTIBULITIS

79%

Decreased itching

79%

Reduction of injuries

URETHRAL BURN

82%

Decreased burning

RECURRENT CANDIDIASIS

94%

Absence of Recurrent Candidiasis after 3 months

83%

Absence of Recurrent Candidiasis after 6 months

VULVO VAGINAL ATROPHY

85%

Decreased burning

81%

Increased fluidity

URINARY INCONTINENCE

81%

Reduction of incontinence

LICHEN

83%

Pain relief

83%

Decreased itching

CARESS FLOW TECHNICAL SPECIFICATIONS

Weight	24 kg
Environment temperature during operation	From 10 to 40 °C
Environment humidity during operation	≤ 80% relative humidity
Environment atmospheric pressure	From 860 to 1060 hP
Supply voltage	220-240 V.a.c., 50/60 Hz
Fuse size	T3,15A, 5x20mm
Electric plug	CEE 7/4 Schuko
Electrical Safety Class	I BF
Medical Risk class	IIA
Power absorbed in input	Max 600W
Oxygen purity	93% +/- 3%
Oxygen flow	2 ±0.2 L/min
Oxygen flow pressure	1±0.1 Bar

DISPOSABLE KIT

Kit Composition	15 Connective Tubes 15 Cannulas 15 Bottles of Caress Flow Solution
Width	18.5 cm
Depth	22.5 cm
Height	17 cm
Connecting Tubes	Anti-crushing tube for oxygen, length 2 m, with male / female connections Length 14.6 cm
Cannulas	Min diameter 9.5 mm Dispensing holes 1.45 mm Syringe hole 0.8 mm
Caress Flow Solution	Bottles of 15 ml of HA with a low molecular weight

CARESS FLOW

OXYGEN FOR VULVO - VAGINAL TREATMENTS

Vestibulitis

Vulvo Vaginal Atrophy

Recurrent Candidiasis

A SYSTEM FOR 6 PATHOLOGIES

Urethral Burn

Lichen

Urinary incontinence

CARESS FLOW SYSTEM



- SAFE**
- PAINLESS**
- ALWAYS REPEATABLE**
- WITH NO SIDE EFFECTS**
- WITHOUT RISKS FOR THE PATIENT**
- WITHOUT RISKS FOR THE PRACTITIONER**



CARESS FLOW SOLUTION



CONNECTING TUBE



CANNULA




AIRBRUSH

Width: 44.4 cm
Depth: 23.8 cm
Height: 58.7 cm

5. LICHEN

For the first 10 minutes oxygen is delivered on the vulva by nebulization (to be performed 2 mm away from the vulva) with low molecular weight Caress Flow Solution (3ml). For the next 5 minutes oxygen combined with Caress Flow Solution is delivered in the vagina (12ml)


5 Treatments
One every week

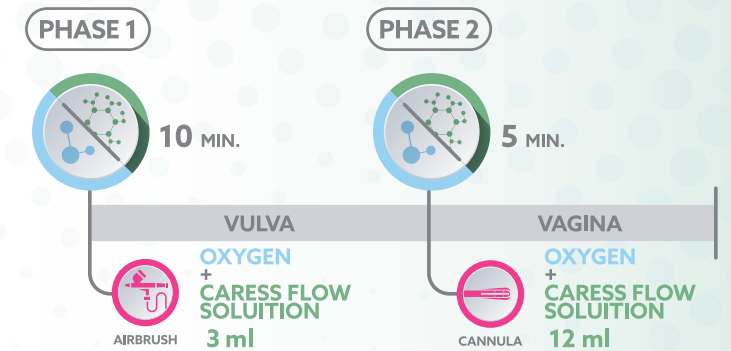

6. URETHRAL BURN

For 20 minutes oxygen is delivered on the vulva by nebulization (to be performed 2 mm away from the vulva) with low molecular weight Caress Flow Solution until the bottle is empty (15ml).

6 Treatments, one every week
+
2 Treatments, one every two weeks

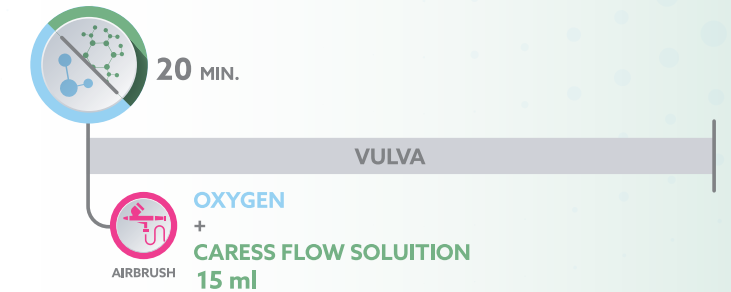


THE PROTOCOLS FOR THE 6 PATHOLOGIES

1. VESTIBULITIS 2. RECURRENT CANDIDIASIS

For the first 5 minutes oxygen is delivered on the vulva by nebulization (to be performed 2 mm away from the vulva) with low molecular weight Caress Flow Solution (3ml). For the next 10 minutes oxygen alone is delivered in vagina and for the last 5 minutes oxygen combined with Caress Flow Solution is delivered always in vagina (12ml).


5 Treatments
One every week

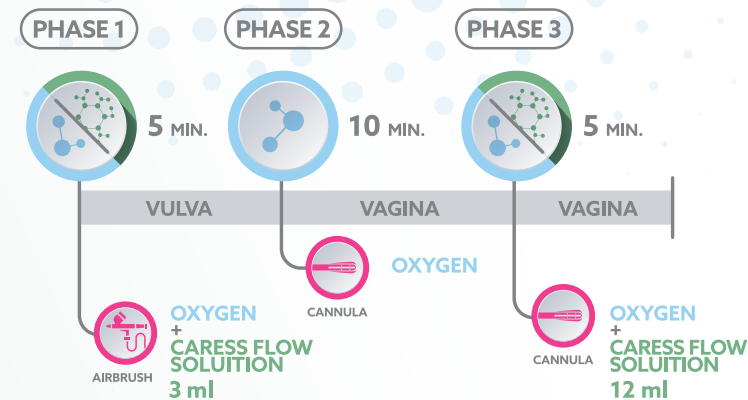

3. VULVO VAGINAL ATROPHY 4. URINARY INCONTINENCE

For the first 10 minutes oxygen is delivered with Caress Flow through the use of disposable cannulas. For the next 5 minutes, oxygen is delivered combined with low molecular weight Caress Flow Solution (15ml).

5 Treatments
One every two weeks

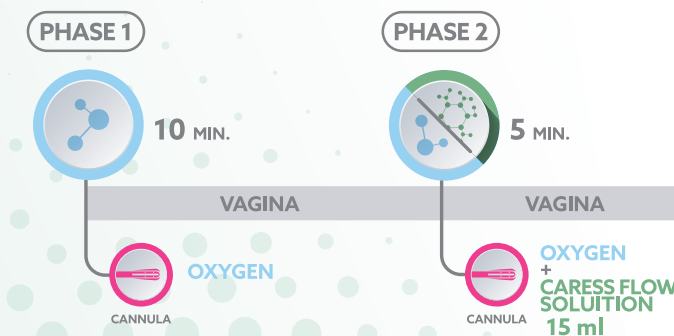

THE SCIENTIFIC DISCOVERY

THE SYNERGY OF 2 NATURAL ELEMENTS

