

INTHERA

Advanced Revolutionary Therapy with RF needle



What is Microneedling RF


Advanced Revolutionary Therapy for All Skin Types with RF needle.

- Micro-needle RF uses microneedles to deliver radiofrequency (RF) energy into varying levels of the skin.
- Applying precisely controlled RF energy directly into the dermis with minimally invasive micro-needles helps to regenerate tissue, revitalize skin for improved texture, firmness, tightening and lifting.
- Fractional Micro-needle RF System can be treated regardless of skin type, even darker skin with a low risk of skin burns and PIH compared with other laser treatments.




How INTHERA Works?


INTHERA is a non-surgical treatment based on the synergy effect of precisely controlled Bi-polar RF + minimally invasive micro-needles.

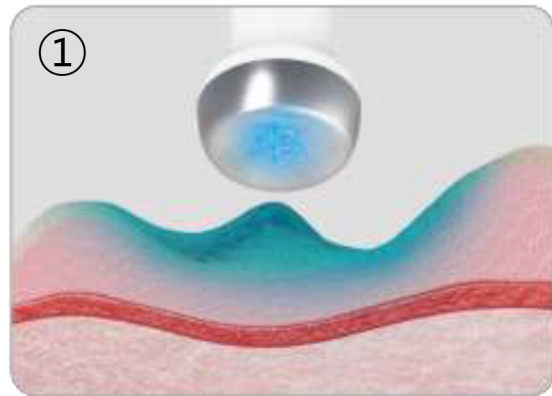

① Before treatment use the cryo handpiece to temporarily anesthetize the skin.

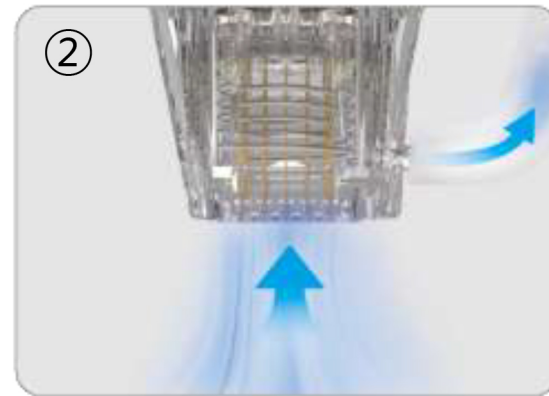

② Place the Micro-needle handpiece on the skin surface and use the vacuum to grip the skin tightly.

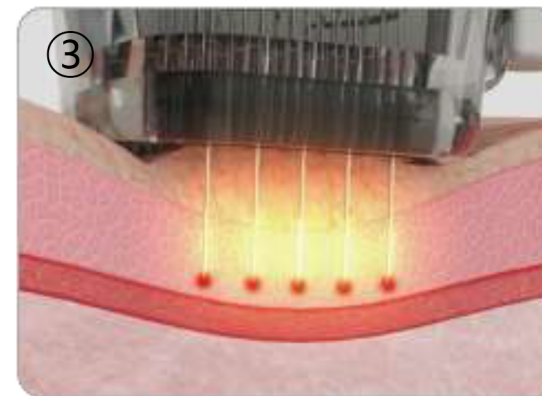

③ Micro-needle penetrate inside of the skin. Bi-polar RF energy is directly applied to the target areas.



④ Collagen regeneration and new elastic production process begins.

Features of INTHERA

Uniquely designed to safely deliver RF energy for optimal results with minimal downtime.

Bi-polar
RF Energy

Insulated,
Non-insulated
Needle


Vacuum
Technology

Cryo
Technology


Bi-polar RF Energy

When micro-needle is injected into the skin, on the same time, Bi-polar RF will be also effects to the skin. The RF energy will be directly affected to the dermis and help regeneration of collagen and elastin.

- Uniform and stable energy
- Minimal energy loss
- Minimal output leaning due to impendence
- Less skin damage
- Less Pain

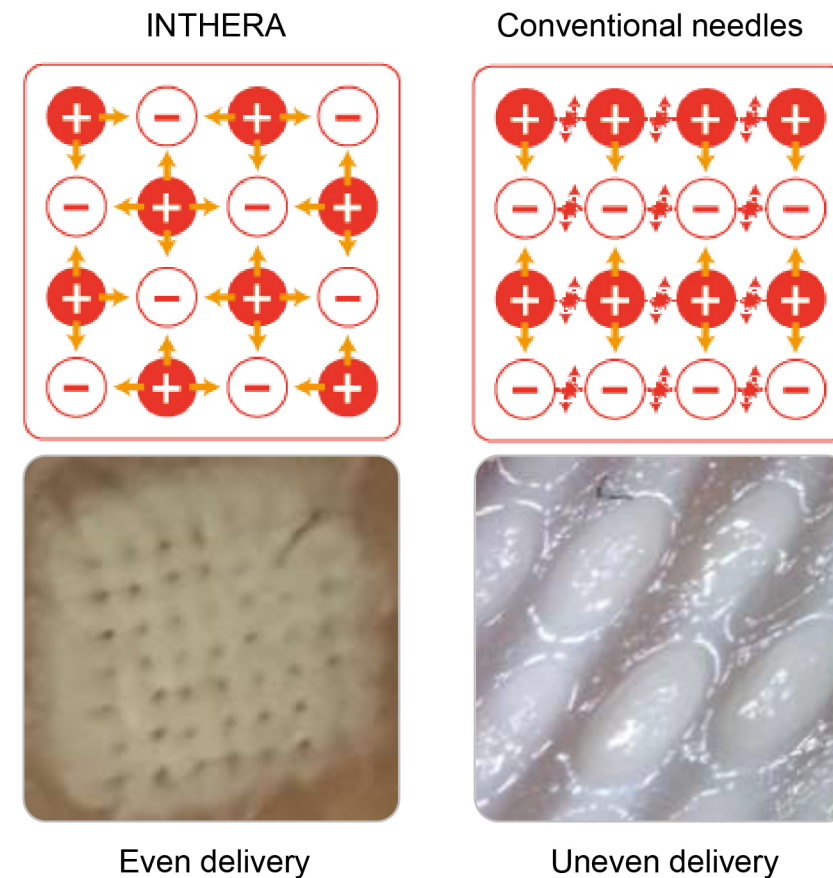


Insulated & Non-insulated Needle

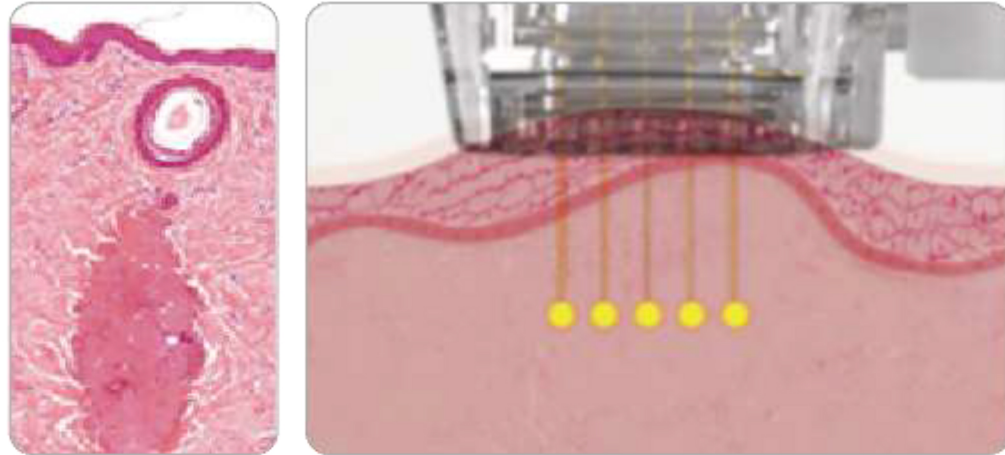

Insulated Needle


- Intensive RF energy transmission
- For **Dermis**
- Less risk of skin surface damage
- Direct energy delivery to the dermis
- No downtime
- Minor bleeding
- Application : wrinkles, elasticity, etc.

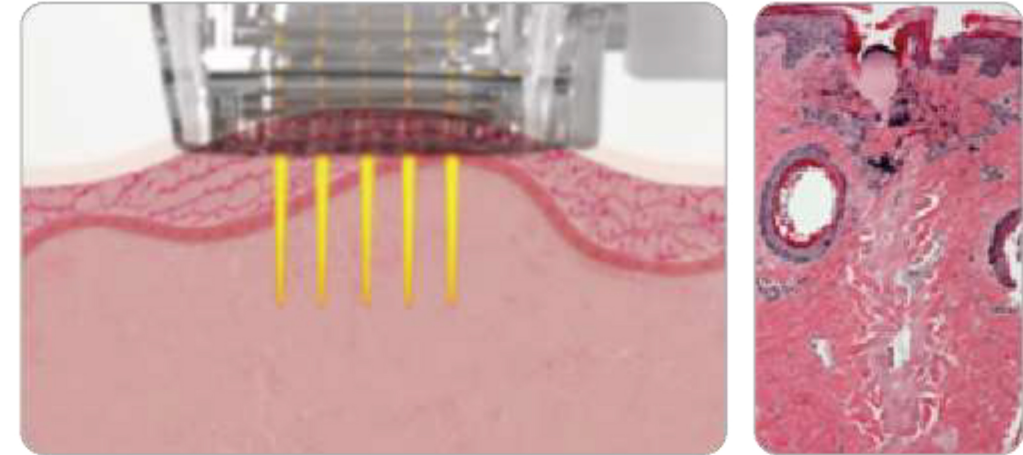

Non-insulated Needle

- Full of RF energy transmission
- For **Epidermis and Dermis**
- Energy delivery including surface
- Deliver more energy (Wide treatment area)
- Short downtime (Scab formation)
- No bleeding
- Application : acne, scar, pore etc.

Needle Type

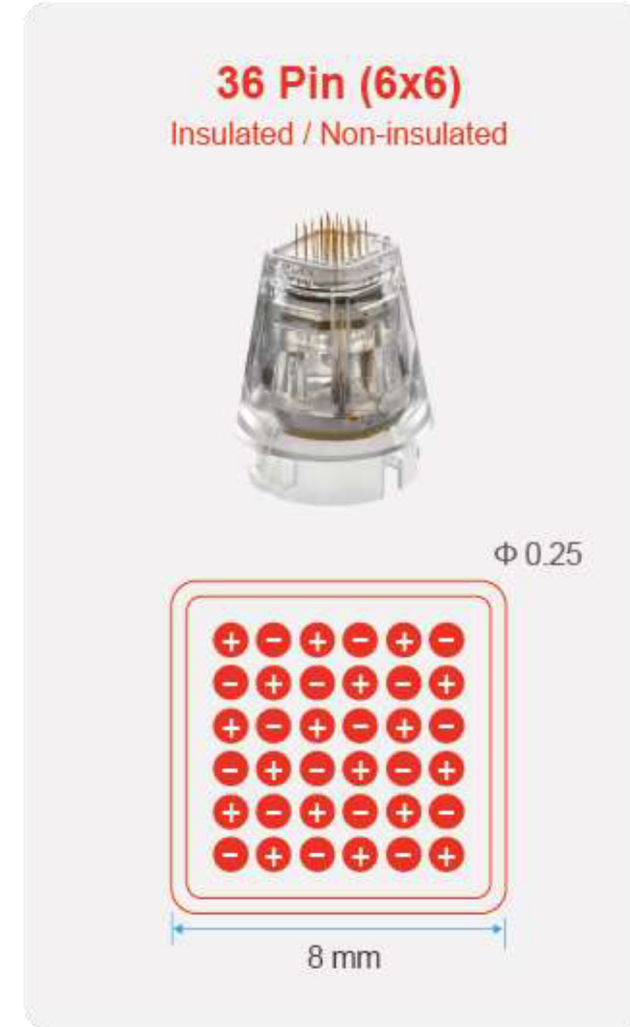

RF Needle with Vacuum Technology

Applies pressure to the skin to evenly insert needles and penetrate more precisely.

- Needles penetrate evenly.
- Minimizes bleeding and pain.
- Perform procedures with less damage.
- Fast and convenient operation.

Optimizing needle For Treat area

Precise depth control from 0.5 to 3.5mm



Cryo Technology

An effective technique to relieve pain, anesthetize skin, and clam the skin.

- Temp Setting : 3 level (0 °C ~ - 10 °C)
- During treatment, The Cryo handpiece is utilized to instantly anesthetize the skin by cooling the epidermis, thereby reducing pain during the treatment.
- After treatment, Cryo handpiece can calm down the skin and reduce swelling, redness.



User Convenience UI Design

Advanced Revolutionary Therapy for All Skin Types with RF needle.



Advantages

Advanced Revolutionary Therapy for All Skin Types with RF needle.

- **Short Treatment Time**

Save time for applying and waiting of anesthesia cream.

- **Minimal Downtime**

Thin needle enables decreased downtime.

- **Safe & Comfortable**

Precisely controlled RF energy is delivered to the target areas.

- **Convenient operation**

Ergonomically designed hand piece provides comfortable grip feeling.



Indication

For all skin types,
even darker skin with low risk of skin burns and PIH.

- Skin texture
- Rejuvenation
- Acne scar
- Wrinkle reduction
- Pore Contraction
- Fine lines Improvement
- Tightening & Lifting
- Neck lines treatment
- Body Scars
- Keratosis Pilaris
- Stretch Marks



Specification

Product Name	INTHERA	
RF Frequency	2MHz, Bipolar	
Vacuum Function	0 ~ 3 Level	
Cryo Function	0 ~ -10 °C	
Depth	0.5~3.5mm	
On Time	100~600ms	
Needle Type	INSULATED	10Pin, 25Pin, 36Pin
	NON-INSULATED	10Pin, 25Pin, 36Pin
Dimension	390 x 500 x 1060mm [W x D x H]	





📍 Sharjah - UAE
☎ +971 6 557 9818
✉ info@maria-trading.com

www.maria-trading.com

[f](#) [t](#) [i](#) [v](#) [in](#) [d](#) /mariatradingco

